

Helpful Hints for Using Your CCTV

1. Do not set up your unit in front of a window.
2. Begin using the unit in intervals of 30 minutes
3. Remember to blink so that you do not get dry eyes from staring
4. It is okay to use artificial tears, use a brand that is preservative – free.
5. Use the bottom or top of your screen as a guide to help you learn how to follow a line.
6. When writing be sure to remember to look at the screen, not the table. It takes lots of practice!
7. If the glare from the screen bothers you, you may want to try a yellow or orange glare control glass.
8. It is okay to work in the dark with your unit.
9. Start with large items first, then work your way to smaller items.
10. Fold the newspaper in half.
11. If you have reading glasses, try using them while working with your unit.

If you have any questions, do not hesitate to call:

Magnifiers & More 440-946-3363